

Multi Drug Test Cup

Test Instructions

1. Person being tested places urine specimen into test cup.
2. Collect a urine specimen directly into the Test Cup.
3. Insure that the sample amount meets the minimum level as indicated on the side of the Cup.

Securely seal lid (That's the end of all urine handling).

4. Wait 2-6 minutes.
5. Read results for each of the 5 drugs on a separate strip for each drug.

Drugs of abuse are detectable:

1. THC Marijuana
2. COC Cocaine
3. AMP Amphetamine
4. PCP Phencyclidine
5. OPI Opiate

Read Results
Allow 5-9 minutes for tests to develop.

