


iCassette™

Drug Screen

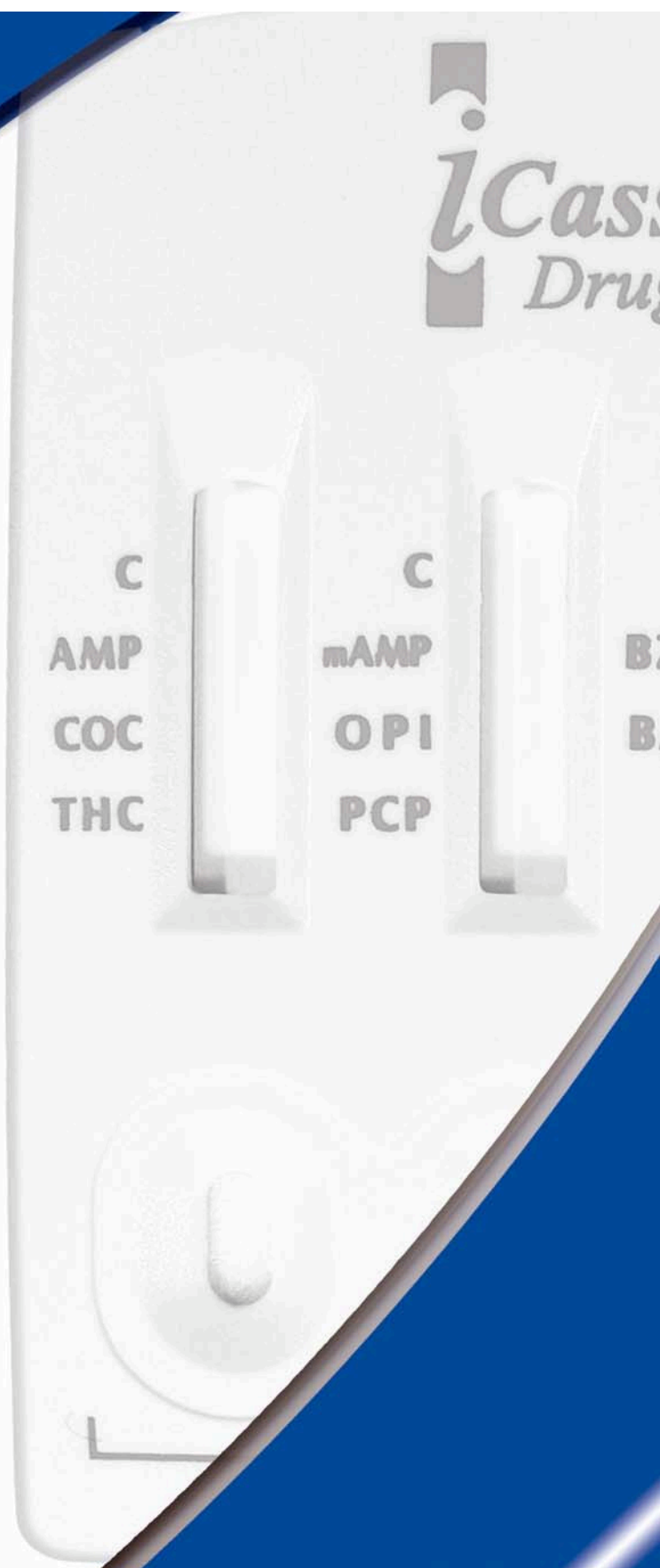

iCassette™ Drug Screen

Corporate Drug Testing Made Simple.

- CLIA Waived panels available
- Runs on 3 mL of specimen
- Laboratory accurate
- Detects 1-12 drugs
- Results in minutes
- Simple procedure

Brands You Can Trust™



iCassette™ Drug Screen

iCassette™ Drug Screen Corporate Drug Testing Made Simple.

Detect the presence of drugs in minutes.
It's drug testing efficiency you can count on.




CLIA Waived iCassette 5 panel shown above.


Drug Name:	Abbreviation:	Cutoff Levels:
Amphetamine.....	AMP.....	1000ng/ml
Barbiturates.....	BAR.....	300ng/ml
Benzodiazepines.....	BZO.....	300ng/ml
Cocaine.....	COC.....	300ng/ml
Ecstasy.....	MDMA.....	500ng/ml
Marijuana.....	THC.....	.50ng/ml
Methadone.....	MTD.....	300ng/ml
Methamphetamine.....	mAMP.....	1000ng/ml
Morphine.....	MOR.....	300ng/ml
Opiates.....	OPI.....	2000ng/ml
Oxycodone.....	OXY.....	100ng/ml
Phencyclidine.....	PCP.....	25ng/ml
Propoxyphene.....	PPX.....	300ng/ml
Tricyclic Antidepressants.....	TCA.....	1000ng/ml

Instructions

- 1** Remove the test device from the sealed pouch.

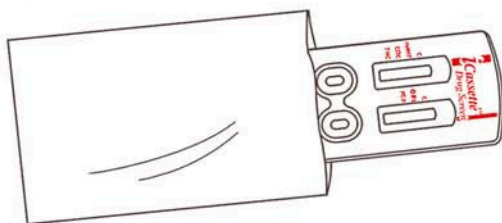

- 2** Hold the dropper vertically and transfer three (3) full drops of urine to the specimen well of the test device.

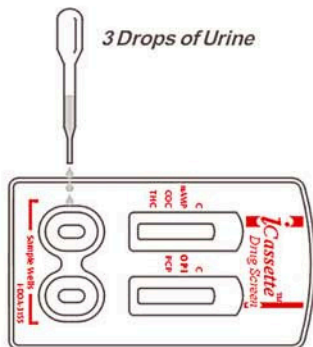

- 3** Read the results at five (5) minutes.
Results stable up to four (4) hours.

