



COLLECTION INSTRUCTIONS

Please read all instructions before collecting your sample.

IMPORTANT REMINDERS: ***This is a pre-paid test; if you lose any portion of this test we are unable to replace it. You are responsible for all shipping fees to send the sample to the laboratory. We highly suggest that you use a traceable method to ship the sample to the lab using the corrugated envelope provided.

Before you begin the collection procedures below please write your name on the transportation vial in the allotted area. Also, be sure to fill out the test request form.

Procedure for collection of sample:

1. Collect urine sample in sterile cup or other sterile receptacle.
2. Using the collection/transportation device, syringe or draw a full tube of sample pulling the syringe handle up.
3. Carefully open the transportation vial and deposit the urine sample by pushing the syringe handle down. Do not overfill the vial.
4. Replace the cap for the transportation vial, replace the vial in the plastic bag, and discard any remaining urine and the syringe.
5. Be sure you have filled out the entire test request form and place the form along with the transportation vial in the corrugated envelope provided.
6. You will need to pay the postage to send the package. We suggest using a traceable method to ensure your sample is delivered. If you do not choose to use a traceable method and the lab does not receive your sample we cannot replace the test.